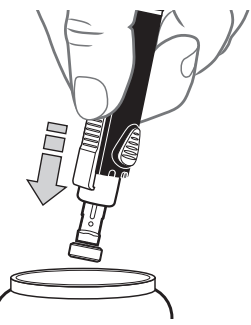
3. Point the device toward a container for sharp objects (an empty plastic bottle with a lid, for example). Label the container properly and keep the container out of the reach of children.
4. Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back.


5. Attach and **lock** the endcap. Twist the white band to lock.

Get the Blood Drop - Palm

ALTERNATIVE SITE TESTING (AST) PALM

Use the **clear** AST endcap.

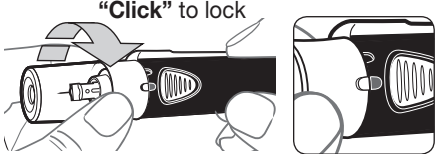1. Insert the lancet as described in *Prepare the Lancing Device*.
2. Attach and lock the **clear** AST endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.


CAUTION: Do not use the clear AST endcap on your fingertips.

3. Select a puncture site from a fleshy area on the palm. Avoid veins, moles, bones, and tendons.


4. Pull back the re-set handle to prepare the lancing device. The release button is raised when the lancing device is ready.

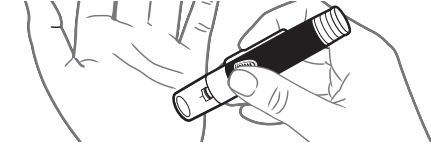5. Press the lancing device firmly against the puncture site, then press the release button UNTIL THE DEVICE CLICKS.


(If the device does not puncture, reset by repeating Step 4.)

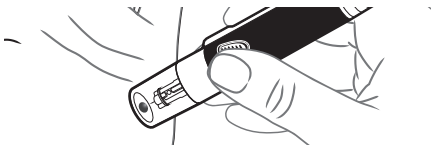6. Do not take the device away from your palm. Maintain steady pressure until a small, round blood drop forms under the clear endcap.

7. Lift the lancing device straight up and away from the skin without smearing the blood drop.

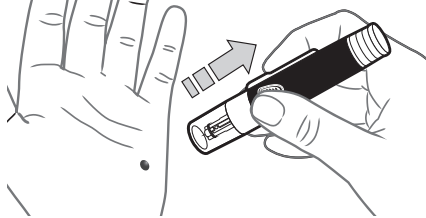8. Test immediately.

Do not test the blood sample from palm if you get:

- Smear blood
- Runny blood
- Clotted blood
- Clear fluid mixed with the blood

9. Wipe the puncture site with a clean, dry tissue and maintain pressure on the site until the bleeding stops.

See: **Eject / Dispose of the Lancet**

Cleaning

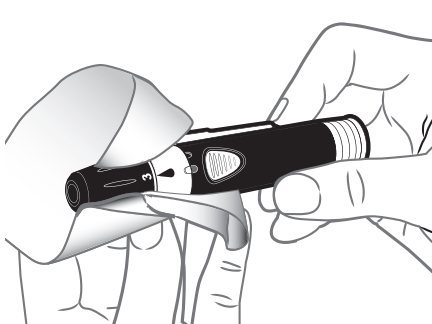
Clean your lancing device once a week. Use a cloth dampened with 70% isopropyl alcohol (found in pharmacies). Do not use bleach (sodium hypochlorite).

Supplies Needed for Cleaning:

- A cloth dampened with 70% isopropyl alcohol (found in pharmacies).
- Do not use bleach (sodium hypochlorite).
- Paper towels

1. **Clean the lancing device** with isopropyl alcohol until all soil is removed.

2. Dry as necessary with a clean paper towel.

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Ascensia, the Ascensia Diabetes Care logo, Contour, Glucofacts, Microlet, the No Coding logo, and Second-Chance are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

All other trademarks are the property of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.

© 2020 Ascensia Diabetes Care Holdings AG. All rights reserved.
www.diabetes.ascensia.com

Ascensia, el logotipo Ascensia Diabetes Care, Contour, Glucofacts, Microlet, el logotipo No Coding y Second-Chance son marcas comerciales y/o marcas registradas de Ascensia Diabetes Care Holdings AG.

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Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

ASCENSIA
Diabetes Care

For information on patents and related licenses, see: www.patents.ascensia.com

90006314/90006490 Rev. 07/20

Ascensia Diabetes Care Spain, S.L.
WTC Alameda Park Edificio 6
4ª planta, Plaça de la Pau, s/n
08940, Cornellà de Llobregat
Barcelona / España
Tel.: 900 100 117
atencioniabetes@ascensia.com

Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

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Microlet®
next
Lancing Device

INSTRUCTIONAL INSERT

- Lancet ejector for safe lancet disposal.
- **Black/white** adjustable endcap for fingertip testing only.
- **Clear** AST endcap for alternative site testing only.
- The MICROLET®NEXT lancing device and MICROLET® lancets are used to draw capillary blood from the fingertip or palm for blood glucose testing.

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WARNING

- Ask your health care professional if Alternative Site Testing (AST) is right for you.
- Do not calibrate a continuous glucose monitoring device from an AST result.
- Do not calculate an insulin dose based on an AST result.

Do not use AST under the following conditions:

- If you think your blood sugar is low (hypoglycaemia).
- When blood sugar is changing rapidly (after a meal, insulin dose, or exercise).
- If you are unable to feel symptoms of low blood sugar (hypoglycaemic unawareness).
- If you get alternative site blood sugar results that do not agree with how you feel.
- During illness or times of stress.
- If you will be driving a car or operating machinery.

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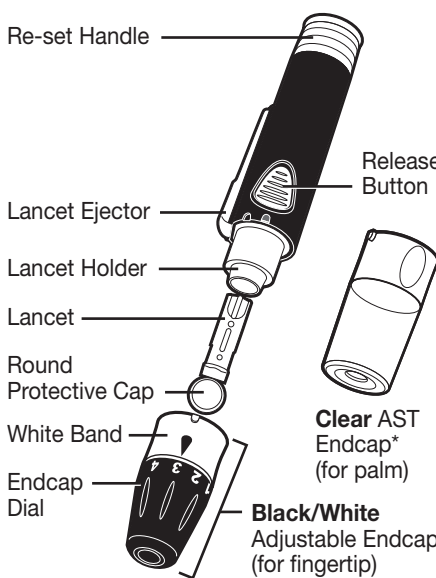
POTENTIAL BIOHAZARD


- The MICROLET®NEXT lancing device is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.
- Do not reuse lancets. Used lancets are not sterile. Use a new lancet each time you test.
- Always dispose of used test strips and lancets as medical waste or as advised by your health care professional.
- Always wash your hands with soap and water and dry them well before and after testing or handling the meter, lancing device, or test strips.

* May not be included in all kits. Available on request from **Customer Service**.

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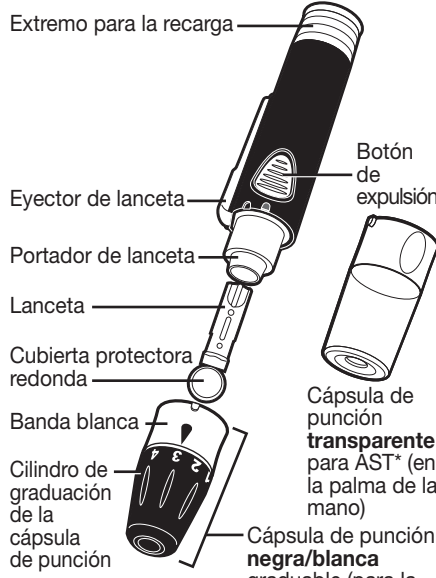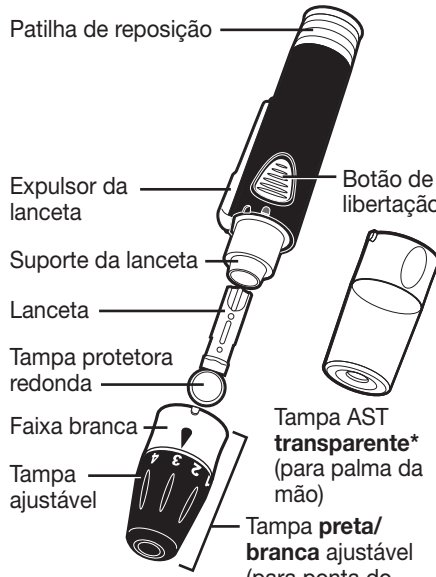
Prepare the Lancing Device

Warning: Keep out of reach of children. This kit contains small parts that could cause suffocation if accidentally swallowed.

1. Wash hands and the puncture site with soap and warm water. Rinse and dry well.

2. Unlock the endcap. Twist the white band so the white raised mark and black indented mark line up as shown.

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

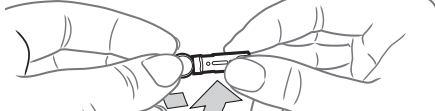
4. Loosen the round protective cap on the lancet by rotating the cap ¼ turn, but do not remove the cap.

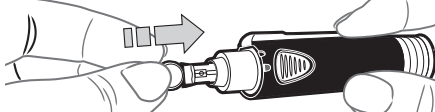
5. Insert the lancet firmly into the lancing device. Push until the lancet comes to a complete stop.

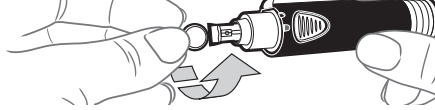
6. Carefully twist off the round protective cap. Save the cap for disposing of the used lancet.

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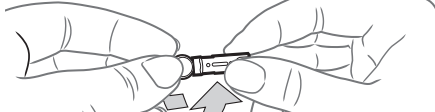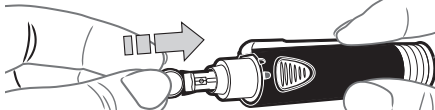
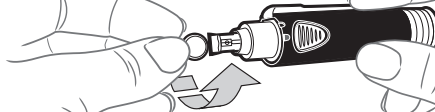
1. Attach the black/white adjustable endcap.
2. Lock the endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.

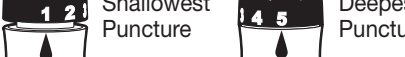3. Rotate the endcap dial to adjust the puncture depth. The amount of pressure applied to the puncture site also affects puncture depth.

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Eject / Dispose of the Lancet

Always dispose of the used lancet as medical waste to prevent injury or contamination to others. Use a new lancet each time you test. **Do not reuse lancets.**

Your MICROLET NEXT lancing device has a lancet ejector. Do not use your fingers to remove the lancet from the lancing device.

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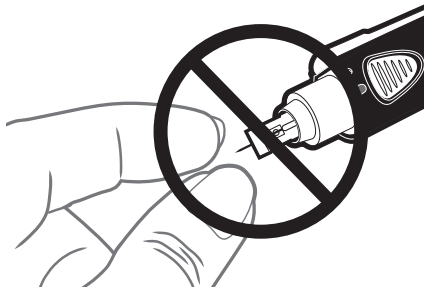
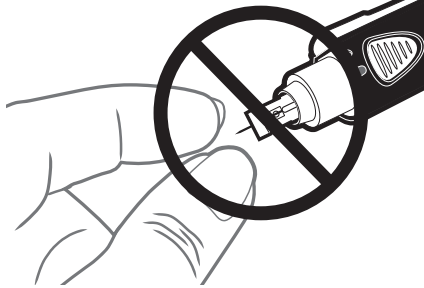
1. Carefully remove the endcap from the lancing device. Twist the white band so the white raised mark and the black indented mark line up. Pull off the endcap.

2. Place the round protective cap that you saved onto a flat surface. With the used lancet still in the lancing device, push the lancet into the center of the cap.

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